

# Important notes

## The return transport - one call is enough!

The return transport is carried out by our logistics partner.

Please observe the export regulations of your country.

GO! Express & Logistics GmbH

Kunden-Nr: 103977

Phone Germany: 0800 1112828

Phone abroad: +49 7131 155650

## Please note on the day of sampling:

- Registration of the collection until 12 o'clock at the latest
- Collection takes place until 2 pm
- Last possible return day is always Thursday
- No collection before holidays

## Content:

- 20 Rodac plates for the examination of surfaces
- 2 bioindicators with spores of the test germ (*Geobacillus stearothermophilus*)
- 1 Transport control bioindicator
- 1 Bottle of hand sanitizer
- 1 Cooling element
- 1 waterproof pen
- 1 Sampling plan

## Do you have any questions:

Write to us at:  
[info@hymo-box.de](mailto:info@hymo-box.de)



For more information,  
please visit:  
[www.hymo-box.de](http://www.hymo-box.de)

Hygiene monitoring made easy

# Sterilization check

## The HyMo- Box

Take your hygiene monitoring into your own hands with the HyMo-Box!

This HyMo box provides you with reliable findings on the function of your steam sterilizer. The test complies with the specifications of the DIN EN ISO 17665 standard - so that your sterilized items are safe.

Using the established swabbing method, our accredited microbiology laboratory determines the hygiene status of your working environment.

Please note the instructions for return transport on the reverse side.



# Step by step guide

for the HyMo box sterilization check

Your advantages: Biological testing can be used to monitor the performance of your sterilizer and the cleaning and disinfection measures in the environment. It's all inclusive - there are no hidden costs for you. Our DAkkS accreditation guarantees you the highest quality of our services.




1

- Open HyMo-Box.
- Place cooling element in the refrigerator.
- In case of a later sampling you can take the box with the RODAC plates and the bag containing the bioindicators the bioindicators in the refrigerator.



2

- Define the day of performance of the tests and accordingly organize return transport for the same day.
- **ATTENTION:** Registration of the return transport must be made before 12 noon, the collection of the shipment will be made by 2 pm, no return transport on Fridays.

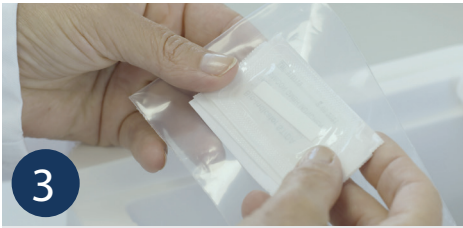


3

- Open HyMo box & place cooling element place in the refrigerator.
- Remove the bioindicators from the packaging, label them and place them in the steam sterilizer with the items to be sterilized.



4

- The bioindicators are to be placed at two different locations close to the sterilized load.
- The transport control remains untouched in the box.
- Start or run through the device.
- After opening the device, transfer the bioindicators back into the packaging.

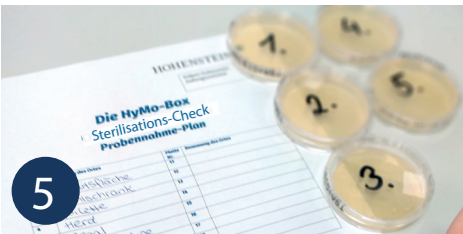

5

- For surface testing, define sampling locations and enter them in the sampling plan.
- Label the bottom of each RODAC plate according to the sampling plan with the corresponding corresponding numbers.



6


#### SAMPLING:

- Disinfect hands.
- Open the lid of a RODAC plate.
- The culture medium surface for for approx. 5 seconds under slight Press on the area to be examined - do not move the RODAC plate.
- Close the lid again.



7

- Proceed with all further RODAC plates according to your sampling plan.
- Stack the RODAC plates back into the box and close it.

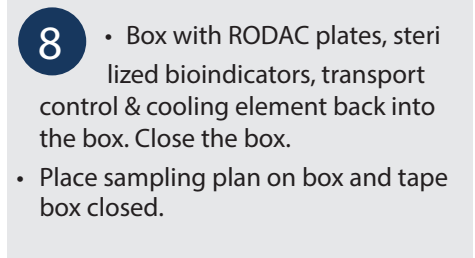

8

- Box with RODAC plates, sterilized bioindicators, transport control & cooling element back into the box. Close the box.
- Place sampling plan on box and tape box closed.